

Skinvisc

- Intra dermal injections must be performed on clean skin (exempt of make-up) using the same precautions as any other intra-dermal filler. Sensitive skin may be pretreated using a local anesthetic cream
- 28 or 30 Gauge needle should be used.
- 2 mL of solution contains 40mg of Hyaluronic acid (MW: 1500-2000KDa) in phosphate buffer and water for injection.
- Intended use:
- Intra-dermal injections for hydration of dehydrated and wrinkled skin tissues (SkinVisc).
- Intra dermal injections must be performed on clean skin (exempt of make-up) using the same precautions as any other intra-dermal filler. Sensitive skin may be pretreated using a local anesthetic cream.
- Total Number of Session: 4
- Frequency: Once in 2 months